




Step 1, Before surgery, Adjust back to desired size using the fastener tabs

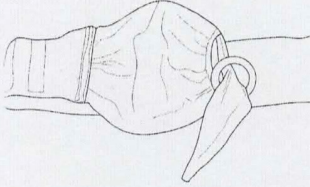


Step 2, On the side your surgery will occur, close and secure the bra cup using the fastener tab

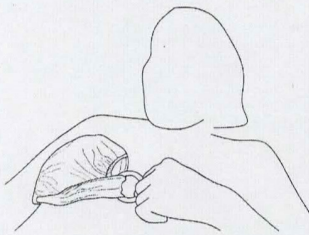


Step 3, Lay bra open on flat surface and recline. Reaching across with your unaffected side, slide the fastened cup gently under your surgically immobile arm; Grab the center ring with your unaffected side and pull to the middle of your chest.

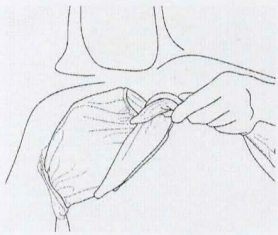

Step 4, Using your unaffected side again, close other bra cup using the fastener tab and pull tight for a good fit.

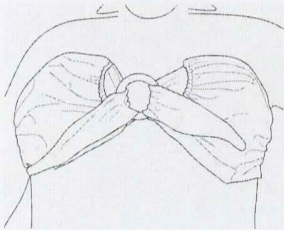

Step 5, Bra should look like this once properly fastened.

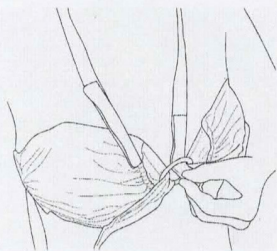

Step 6, Stand up and thread the optional neck strap with unaffected side, if extra support is desired.